PHI Primary
Healing
Implant

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KRISTAL
IMPLANTOLOGY - ORTHODONTICS

Kristal S.r.l. via Edison, 14
20090 Trezzano S/N (MI) - Italia
Tel. +39 024453401 - info@kristalrsl.net

- ① ISTRUZIONI PER L'USO DELLE FRESE LINEA PHI
- ② INSTRUCTIONS FOR USING THE PHI DRILLS
- ③ MODE D'EMPLOI DES FRAISES DE LA LIGNE PHI
- ④ GEBRAUCHSANLEITUNG FÜR DIE BENUTZUNG DER FRASEN PHI
- ⑤ INSTRUCCIONES PARA EL USO DE LAS FRESAS DE LINEA PHI

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② INSTRUCTIONS FOR USING THE PHI DRILLS

Description

Cutters and scalpels contra handpiece and reamers of PHI line are made of stainless steel, Taps manual are made of Titanium Grade 5 (Ti6Al4V) per ASTM F136. All drills are packaged in double-container unless they are marketed within the Surgical Tray. The first pack is a blister PET thermally sealed with medical paper, the second is a cardboard container in which the immediate packaging is placed. Both packs are labeled with the identification code and the lot number. The product is supplied non-sterile clean.

Surgical instruments to PHI mark meets the essential requirements of Directive 93/42/EEC and subsequent amendments concerning medical devices.

Indications

The opercolatore scalpel (mucotome) serves for performing a circular incision into the gum (gingival operculum) which permits subsequent removal of the residual gum engraved and the exposure of the bone portion through which it will perform the operation of the endosseous implant prosthesis.

The initial cutter (for corticotomy) serves for the depth of implant cavity for the management of cervical cortical bone portion.

The probe cutter serves for the exploration of the cavity and to produce a guide tunnel for the stepped cutters.

The cutters in steps are used for drilling the bone intended to receive only the facilities of the PHI line. The manual osteotome is used to refine the bone ca-

lity transforming it from a step in a cylindrical cavity and frusto-conical.

The taps are used to create a spiral site for the screw fixing of PHI plants in the presence of compact bone. Before use, it is recommended to arrange for sterilization. The method of sterilization is recommended nell'imbustare the instrument or store it in Tray Surgical, then autoclave for 30 minutes at 120/130°C.

During surgery struementi storing in a jar containing sterile saline.

After surgery:

- Run a wash cycle with detergent solution in an ultrasonic bath or washing tools;

- Carry out a visual inspection by removing any debris with a soft brush. The cleaning process of the gingival punch is simplified by the ability to completely remove this cutter;

- Rinse the drills in demineralized water, preferably in a glass immersed in the ultrasonic bath (attention cutters must not be in contact with each other);

- Proceed to dry the drills with a clean air jet;

- Storing the drills in Tray Surgical sterilization;

The rotation speed of the milling cutter for its characteristics must be very small (30-70 rpm); if the bone is fragile and delicate, the number of turns must be further lowered (up to 10-20 rpm). The cutters can work at a rotation speed of up to 40 rpm with no irrigation, 70-100 rpm in the presence of irrigation. It is preferable to use handpieces contra with high reductions.

Applications

The ability of the cutters cut is limited to 30-40 applications. The use of a milling cutter with worn cutting edges can cause more effort during use and overheating the bone compromising the operation results.

Contraindications

Lack of adequate bone support.

Warnings

The drills of the PHI line can only be used by dentists, dental surgeons, maxillofacial surgeons.

The cutters of the PHI line must be used with a back and forth motion, frequently removing from the cavity to allow the cores to escape.

- For irrigation use a saline solution at low temperature (4-5°C) to prevent damage due to overheating of the bone.

- For washing does not use hydrogen peroxide (H2O2), since tools would lose their cutting capacity.

- Handle the cutters only with sterile tweezers and wearing powder-free gloves (it takes into account the latex in case of sensitive or allergic patient) not to contaminate it.

- Place the double-bagged Cutter holder and sterilize according normal cycle for surgical instruments.

- During sterilization it is necessary to store the cutters so that they are not in contact with each other.

- The drills should be stored until they are needed in the original container in a dry place at room temperature.

- To clean the tools use only products suitable for surgical instruments.

- To replacements you must warn by phone or by e-mail within 10 days of receipt of goods; making the return of the product in its original filling a regular transport document of the commodity and the authorization request document made that comes from your target agent. Returns are not accepted tampered with or damaged packages.

Known characteristics and technical factors known to the manufacturer that could cause danger

Production defects, inadequate mechanical characteristics.

Precautions

If the injury or iatrogenic injury occurs at particular anatomical structures (nerves, adjacent teeth, maxillary sinuses, etc.) The following may be a reversible or irreversible damage to these structures. It is recommended to always use the drill probe before milling steps.

Supply

Box: no. 1 units.

Tray Surgical: complete set of cutters dedicated to the specific line.

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